


Procedure for hydrating. - Enhancing handling characteristics

1. Open

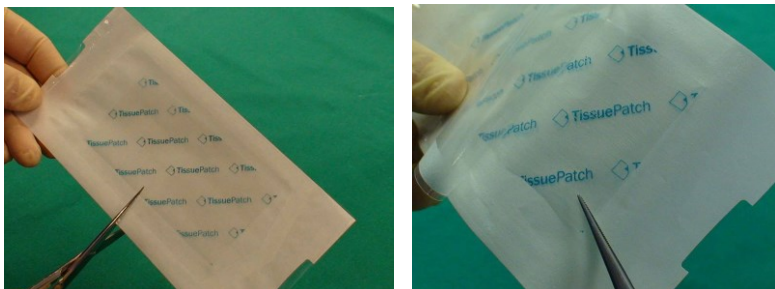

If you need to cut TissuePatch™ to size do so before opening the pouch.

Peel apart the inner pouch and remove with forceps.

2. Hydrate

In a suitable sized kidney bowl, with the 'TissuePatch' logo readable, place the patch between a surgical swab.

Saturate the swab with sterile saline/water - ambient or up to 37°C.

TissuePatch™ can be used from 1 minute after hydration.



3. Apply

Keep the patch moist between the swab until needed.

Use within 30 minutes after hydration.

Pick up using forceps and apply to the target tissue as directed in the Instructions for Use.

Always ensure you can read the 'TissuePatch' logo.

