TissuePatchTM 3

the rapid barrier

Surgical Sealant and Barrier

Kirurgisk forsegling og barriere

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GB

TissuePatchTM – **Surgical Sealant and Barrier**

Multilaminate adhesive, sealant and barrier film for adjunctive prevention of air, blood and/or fluid leakage.

Instructions for Handling and Use

DESCRIPTION
TissuePatch3 is a STERILE, self-adhesive, absorbable surgical sealant and barrier. The product is a clear film with a blue TP 3 logo visible through the patch. TissuePatch3 is available in four different sizes summarised below.

Size	Ref. No.
50 × 25 × 0,04mm	TP3-01
50 × 50 × 0,04 mm	TP3-02
50 × 100 × 0,04 mm	TP3-03
100 × 100 × 0,04 mm	TP3-04

For each product size, a single device is packaged within a sterile inner pouch, which is further sheathed in an outer pouch of the same film which is NON-STERILE. TissuePatch3 is for single patient use only and once opened may not be resterilised. Product should not be re-used as this presents numerous hazards including the risk of cross contamination and risk of stenily.

COMPOSITION

TissuePatch3 is a four layer device comprising alternate layers of poly(lactide-co-glycolide) and poly(N-vinyl-pyrrolidone-co-acrylic acid,-co-N-hydroxysuccinimide ester of acrylic acid,) "Terpolymer". TissuePatch3 contains <0,01% w/w Methylene Blue.

STORAGE

TissuePatch3 is to be stored at 2-25°C. The product should not be frozen. The use by date of this product is printed on the packaging.

MODE OF ACTION
TissuePatch3 is a multilayered product. The poly(lactide-co-glycolide) films act as a cohesive matrix within the patch and as an anti-adhesion barrier film on the patch's top surface. The film of Terpolymer becomes adhesive when applied to tissue and immediately adheres to the surface by ionic and covalent interactions and acts as a sealant to prevent leakage. The patch remains in position while it slowly degrades until substantially absorbed in 50 days.

INDICATIONS
TissuePatch3 is indicated for use to seal and reinforce against air leakage in thoracic surgery and against leakage of low pressure or oozing bleeding or fluid leakage following surgical procedures on soft tissue. TissuePatch3 is intended for use as an adjunct and is not intended to replace sutures or staples, as appropriate, in tissue approximation.

CONTRA-INDICATIONS
The use of TissuePatch3 must be considered on an individual basis. It should not be used on patients with a known allergy or sensitisation to its constituent components.

NOT FOR EXPORT TO U.S.A

Direct all correspondence to your local distributor

NOT for intravascular use.

WARNINGS AND SAFETY INFORMATION
TissuePatch3 should not be used as the primary means of support. The product is to be used as an adjunct, providing temporary support.

TissuePatch3 is not a thrombogenic haemostat. It should not be used to control arterial bleeds; it can however be used adjunctively to control low pressure/oozing bleeding.

TissuePatch3 should not be used on mucosal surfaces.

TissuePatch3 should not be used on the skin.

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